


What will you need?

- The drug being tested
- Soap wipe
- Clean cup or cooker (holds 5mL)
- Water (5mL)
- Micro-scoop
- BTNX fentanyl test strip
- A timer, watch or phone

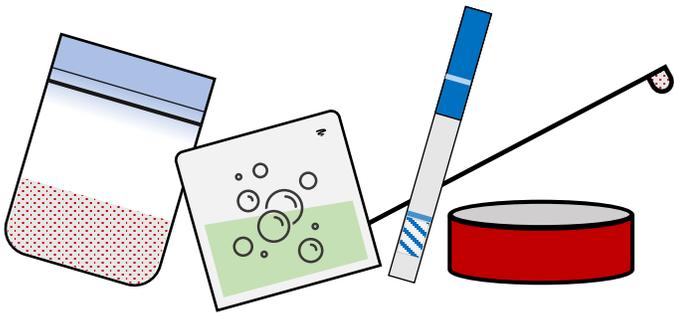

Use the QR code to view
the full instructions.



! CAUTION


- Test strips are never 100% accurate.
- A negative test result does not mean that your drug is safe to use.
- Do not use tests that are expired or tampered with. Dispose of compromised tests normally.
- Humidity & temperature can impact results. Bring test strip to room temperature before use.

Are you testing meth or MDMA?

False positive results can occur with some stimulants. Double-check your result. Test the same sample again: add 30 mL of water & use a new strip.

What will you need?

- The drug being tested
- Soap wipe
- Clean cup or cooker (holds 5mL)
- Water (5mL)
- Micro-scoop
- BTNX fentanyl test strip
- A timer, watch or phone

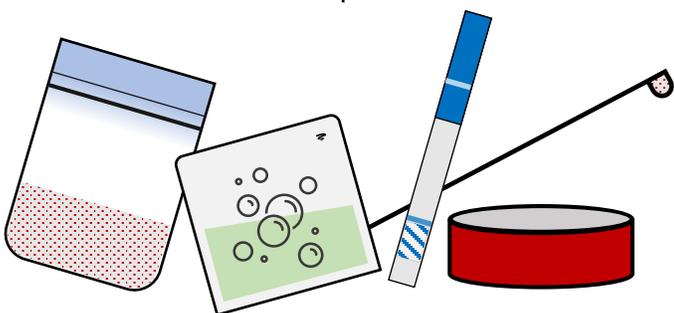

Use the QR code to view
the full instructions.



! CAUTION

- Test strips are never 100% accurate.
- A negative test result does not mean that your drug is safe to use.
- Do not use tests that are expired or tampered with. Dispose of compromised tests normally.
- Humidity & temperature can impact results. Bring test strip to room temperature before use.

Are you testing meth or MDMA?


False positive results can occur with some stimulants. Double-check your result. Test the same sample again: add 30 mL of water & use a new strip.

Results are never 100% accurate. A specific result does not mean that your drug is safe.
Avoid using alone, always go low and slow.

Visit a safe consumption site or call NORS 1-888-688-NORS (6677) before using.


STEP 1 - Crush & Mix

Clean your hands/ surface. Crush and mix the drug. Break up clumps

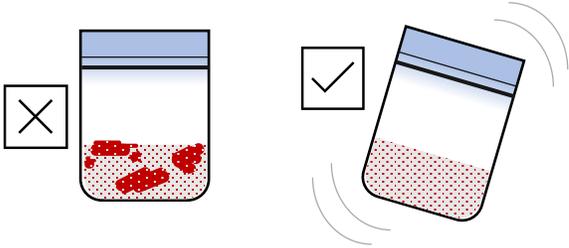


STEP 3 - Test

Hold the test strip by the blue end. Insert the white end into the solution. Hold the strip there for 15 seconds.

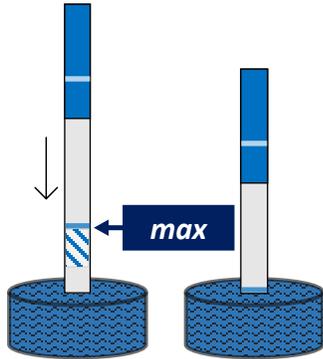


STEP 2 - Measure & Stir

Fill a clean cooker or cup with 5mL (1 ampule) of water. Add 1 micro-scoop of the drug to the water. Stir.

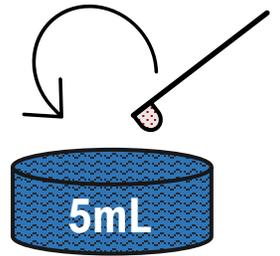

STEP 4 - Results

Wait 5 min & read result. Do not read result after 10 min.

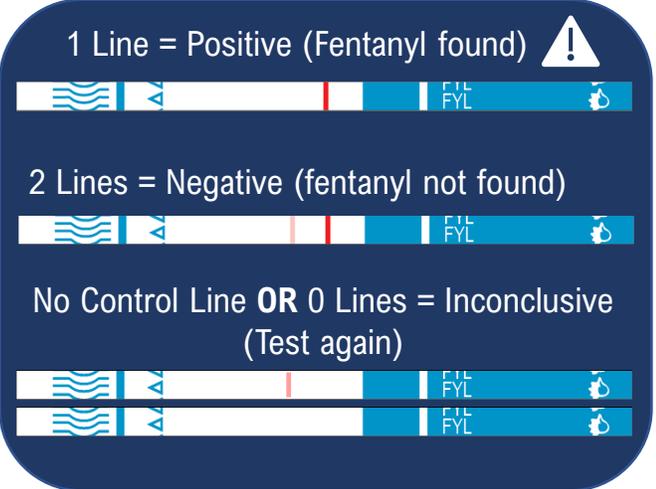


Results are never 100% accurate. A specific result does not mean that your drug is safe.
Avoid using alone, always go low and slow.

Visit a safe consumption site or call NORS 1-888-688-NORS (6677) before using.


STEP 1 - Crush & Mix

Clean your hands/ surface. Crush and mix the drug. Break up clumps

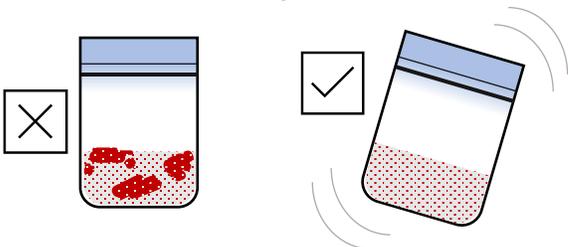


STEP 3 - Test

Hold the test strip by the blue end. Insert the white end into the solution. Hold the strip there for 15 seconds.

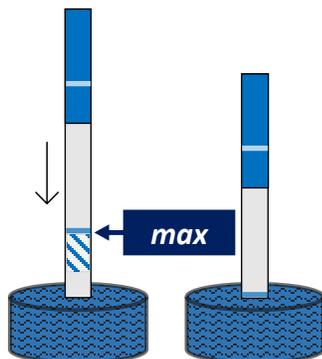

STEP 2 - Measure & Stir

Fill a clean cooker or cup with 5mL (1 ampule) of water. Add 1 micro-scoop of the drug to the water. Stir.

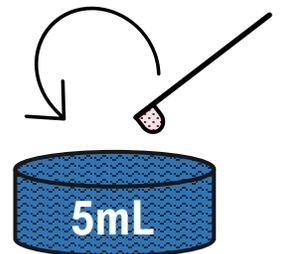

STEP 4 - Results

Wait 5 min & read result. Do not read result after 10 min.

